

Instructions for Humphrey FDT (Field Test Machine)



1. Lift off Top Foam Packing
2. Carefully lift out machine
3. Remove and save red plastic bag and silica gel pack (to keep humidity out)
4. Place FDT on table facing person
5. Plug in clicker on bottom of machine
6. Plug in power cord and always use surge suppressor
7. Turn on machine. Power switch next to cord plug.

Operating Instructions

1. Push top button (run patient tests)
2. input age (+10, -10, +1) then push "accept setting"
3. Instruct screenee:
 - a. Show laminated sheet.
 - b. Keep glasses on (unless only used for reading)
 - c. Say: "When you look into the screen, you will see a white background with a black dot in the center. Try to look only at the black dot. The machine will show you a series of flickering lines around the screen. Push the button when you see a flicker, but do not look around for them. Only look at the black dot."
4. Make sure head is all the way into machine
5. Push top button: "Screening test"
6. When done with right eye move eye piece over for left eye and start by pushing top button
7. When done, machine will print out results
8. If there are more than 2 squares darkened, person should be advised to seek further care

To pack up

1. Unplug power and clicker cords
2. Place machine in bag with silica
3. Place machine in foam in carton, then put cords and surge protector in
4. Put instruction sheets in
5. Cover with top foam piece

Problems or questions call (330) 823-1680 and ask for Dr. Lehrer or Dr. Young.
If after hours or on weekends ask Answering Service to page Dr. Lehrer